NSN 6515-01-338-3254

Surgical Instruments Lubricant - Page 1 of 1



View Online at https://aerobasegroup.com/nsn/6515-01-338-3254

Special Features:

Mix one part concentrate with five parts of distilled water to produce an immersion bath; immerse instruments in bath for approx. 30 to 45 seconds; remove and let drain for approx. 30 seconds, do not rinse or wipe instruments; weck-lube is not a detergent or a rust remover, it is intended to lubricate and provide corrosion control

Shelf Life:

N/a

Unit Of Measure:

Demilitarization:

No

Fiig:

T133-c