

View Online at <https://aerobasegroup.com/nsn/6515-01-600-9429>

**Product Name:**

Ostene soluble bone haemostasis material

**Special Features:**

Does not stay in the body. Creates a mechanical barrier to stop bleeding during the operating procedure. Dissolves and is completely excreted from the body. Dissolves completely from the site of application typically within 6 to 48 hours. Used straight from its sterile foil packet. Worm by kneading in the dry, sterile gloved hand and then apply by pressing into the bone. Putty like consistency when wormed to body temperature. Apply to a thickness of 1 to 2 mm. 2.5g bar for general surgery. Box of 12 bars (2.5g).

**Shelf Life:**

N/a

**Unit Of Measure:**

--

**Demilitarization:**

No

**Fiig:**

T09700