

View Online at <https://aerobasegroup.com/nsn/6520-01-613-6231>

Usage Design:

Dental restoration

Product Name:

Investment, dental, hs investment 100g envelopes 100s

Special Features:

A universal phosphate-bonded investment suitable for use with all pressable ceramic systems and casting alloys, and both the rush and overnight techniques. Mixing ratio 100g powder to 25ml liquid; mixing time under vacuum 60 seconds; working time 5-7 minutes; setting time approx. 15 minutes. 100 gram packet of powder; package of 100 packets

Shelf Life:

N/a

Unit Of Measure:

--

Demilitarization:

No

Fiig:

T243-d